

Your Eversense E3 CGM System automatically creates 4 reports which you can view at any time right in your app. These reports may help you and your health care provider fine tune your diabetes management plan.

To access your In-App Reports, go to Main Menu > Reports

Glucose Pie Chart report:

Time in Range report:

- Displays time spent within, below and above your personal target range that you set up in your mobile app.
- Is viewable for 1, 7, 14, 30 or 90 days of data.
- Comparing data at different time intervals may help you and your health care provider see how changes to your diabetes care plan are working.



© 2024 Senseonics, Inc.

© 2024 Ascensia Diabetes Care Holdings AG. All rights reserved.

All rights reserved.

- Displays your percentage of time spent in preset glucose ranges, based on the standardized AGP report guidelines.
- Tapping on the information icon displays more information about these fixed ranges.
- Is viewable for 1, 7, 14, 30 or 90 days of data.



Weekly Modal Summary report:

- Provides a summary of your last seven days of glucose information in a 24-hour line graph format.
- Makes it easy to see glucose trends by time of day.
- **Red bars** show amount of glucose variability at a given time.
- Scroll down to see additional glucose statistics based on pre-set time slots.

lucose Within Target Levels

6AM

122

29.5 27.4 54.2 76.4

Average

Std Dev

Reports

Veekly Modal Summary (mg/d (May 18, 2018 - May 25, 2018)

12PM

Statistics

117

6PN

198

124

174

Glucose Statistics report:

- Provides average, low, and high glucose values based on preset time slots.
- Is viewable for 1, 7, 14, 30 or 90 days of data.
- Comparing data at different time intervals may help you and your health care provider see how changes to your diabetes care plan are working.

Glucose Within Target Levels						
Reports						I Û
1 Day	7 Days		14 Days		30 Days	90 Days
Glucose Statistics (mg/dL) (May 24, 2018 - May 25, 2018)						
Perio	od	Av	g.	Low	High	Std Dev
12AM - 6	5AM	12	2	78	190	29.5
6AM - 12PM		113		78	163	26.7
12PM - 6PM		197		93	280	50.0
6PM - 12AM		174		80	286	76.4
ALL		150		78	286	60.6
• • •						

All reports can be viewed, emailed or printed directly from the app!



Distributed by Ascensia Diabetes Care 5 Wood Hollow Road Parsippany, NJ 07054 844–SENSE4U (736–7348) ascensiadiabetes.com/eversense

Manufactured by Senseonics, Inc. 20451 Seneca Meadows Parkway Germantown, MD 20876-7005 USA The Eversense[®] E3 Continuous Glucose Monitoring (CGM) System is indicated for continually measuring glucose levels for up to 180 days in persons with diabetes age 18 and older. The system is indicated for use to replace fingerstick blood glucose (BG) measurements for diabetes treatment decisions. Fingerstick BG measurements are still required for calibration primarily one time a day after day 21, and when symptoms do not match CGM information or when taking medications of the tetracycline class. The sensor insertion and removal procedures are performed by a health care provider. The Eversense E3 CGM System is a prescription device; patients should talk to their health care provider to learn more.

For safety information, see https://www.ascensiadiabetes.com/eversense/safety-info/ Eversense, Eversense E3 Continuous Glucose Monitoring and the Eversense logo are trademarks of Senseonics, Incorporated. Ascensia and the Ascensia Diabetes Care logo are trademarks and/ or registered trademarks of Ascensia Diabetes Care Holdings AG. Android is a trademark of Google LLC. Apple Watch is a product of Apple, Inc., and may be separately purchased from an authorized Apple retailer. Apple Watch is not included with the Eversense CGM System. All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied. PP–SENS–GBL–0099